



Sound That Fits

Helpful Tips For Our Audiologist Partners

1. Our custom in-ear hearing protection and enhancement requires a full shell impression and must include the full helix, crus of the helix, tragus, and antitragus. It is IMPORTANT that the impressions be taken up to the second bend of the ear canal.
2. The impressions must be taken with a high quality low expansion silicone impression material. We suggest TEMPLET which can be purchased from Fluxx Audio Inc.
3. The impressions should be taken with an open jaw. A much better retention is received with an open jaw impression

Use a 1 inch bite block during the impression process.

This stabilizes the process and is more comfortable for the customer.

It also ensures that there is no jaw movement while the material is curing.

www.fluxxaudio.com.com

If you have any questions or concerns please
feel free to contact us @

Fluxx Audio Inc.
PPO BOX 429
Ambridge, PA 15003
PH:1-800-532-7672
info@fluxxaudio.com